

---

# Skin Repair Dressing Instructions

**【Product Name】** Skin Repair Dressing

**【Model Specifications】** 30g x 5 patches/box

**【Product Performance, Structure, and Composition】** The skin repair dressing consists of a medical non-woven fabric and a dressing. The dressing is composed of purified water, carbomer, and trehalose. It is sterilized by irradiation and supplied sterile.

**【Key Performance Indicators】**

1. The cytotoxicity of the materials used in the product should not exceed Class II;
2. The product should not cause allergic reactions or intradermal irritation;
3. Sterility: Sterile after irradiation sterilization.

**【Scope of Application】** For the care of minor wounds, abrasions, cuts, and other non-chronic wounds, and the surrounding skin.

**【Contraindications】**

1. Use with caution in those with allergies;
2. Contraindicated on infections, open wounds, burns, and scalds;
3. Contraindicated in pregnant or breastfeeding women;
4. Use with caution in children.

**【Instructions】** After cleansing, apply the product to the face. Remove after 15-20 minutes and gently massage until absorbed. [Precautions]

1. Do not use this product after the sterilization expiration date.
2. This product is sterile. Please carefully inspect the inner packaging before use. Do not use if it is damaged or contaminated.
3. This product is for single use only. Reuse is strictly prohibited. Please dispose of it after use.
4. This product is for external use. If it gets into eyes, rinse immediately with water.
5. Please read the instructions carefully before use.



One-time use, destroyed after use



Do not use if the packaging is damaged



Sterilized by irradiation

**Storage Conditions and Methods:** Store indoors in a ventilated, dry, light-proof environment with a relative humidity not exceeding 80%, free from corrosive gases.

**【Registrant/Manufacturer】** Changde Yinuo Biopharmaceutical Co., Ltd.

**【Registrant/Manufacturer Address/Manufacturing Address】** Fengkang Avenue,

---

Xicheng Community, Dajinggang Town, Anxiang County, Changde City, Hunan Province (within Jinggang Park)

**[After-Sales Service]** Changde Yinuo Biopharmaceutical Co., Ltd.

**[Production License Number]** Hunan Food and Drug Administration Medical Device  
Production Permit No. 20210068

**[Registration Certificate Number/Product Technical Requirements Number]**  
Hunan Medical Device Registration Permit No. 20212141563

**[Manufacture Date]** See outer box for details

**[Product Expiration Date]** 2 years

**[Product Instruction Manual Revision Date]** April 25, 2021

**Note:** Products without the Changde Yinuo Biopharmaceutical Co., Ltd. traceability code are counterfeit and substandard products.